



FreeStyle
LibreLink

Quick Start Guide for
FreeStyle LibreLink app

A FreeStyle Libre product




Setup Overview

Refer to your User’s Manual for full system instructions and information.



Download **FreeStyle LibreLink**
from App Store

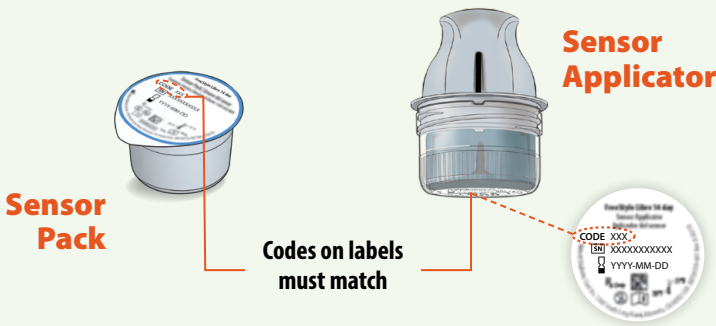


- 1 Assemble and Apply Sensor to the back of your upper arm
- 2 Set up App
- 3 Use the App to scan your Sensor

After the start-up period, you can check your glucose

1 Assemble and Apply Sensor to the back of your upper arm

Get Ready

STEP 1

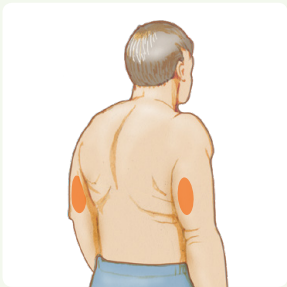


Select site on back of upper arm. Do not use other sites as these are not approved and may result in inaccurate glucose readings.
Note: Avoid scars, moles, stretch marks, lumps, and insulin injection sites. To prevent skin irritation, rotate sites between applications.

STEP 2



Clean site with alcohol wipe. Allow site to dry before proceeding.

STEP 3

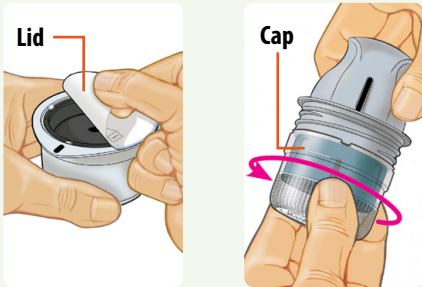


Peel lid completely off Sensor Pack. Unscrew cap from Sensor Applicator.
CAUTION: Sensor codes must match on Sensor Pack and Sensor Applicator or glucose readings will be incorrect.



Prepare Sensor Applicator

STEP 4

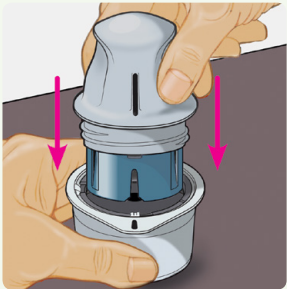

Line up dark mark on Sensor Applicator with dark mark on Sensor Pack. On a hard surface, press down firmly on Sensor Applicator until it comes to a stop.

STEP 5



Lift Sensor Applicator out of Sensor Pack.


STEP 6



Sensor Applicator is ready to apply Sensor.
CAUTION: Sensor Applicator now contains a needle. Do not touch inside Sensor Applicator or put it back into Sensor Pack.


Apply Sensor

STEP 7

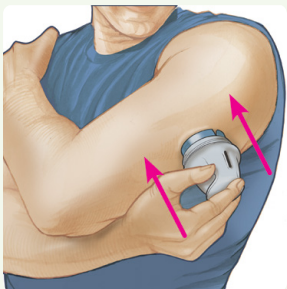

Place Sensor Applicator over site and push down firmly to apply Sensor.
CAUTION: Do not push down on Sensor Applicator until placed over prepared site to prevent unintended results or injury.

STEP 8

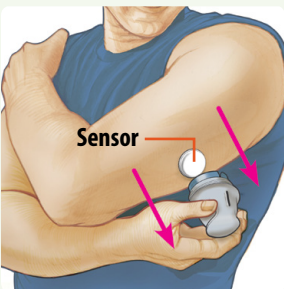

Gently pull Sensor Applicator away from your body.

STEP 9




Make sure Sensor is secure. Discard used Sensor Applicator and Sensor Pack according to local regulations.



2 Set up App

STEP 1

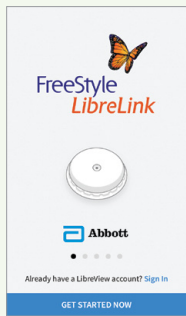

Check your smartphone is connected to a WiFi or cellular network. Tap **Sign In** or **GET STARTED NOW** .

Review the legal information and follow instructions to set up a LibreView account or login to your existing account.

STEP 2



Enter your personal settings. Tap **NEXT** after each screen.



STEP 3



Review important information about glucose readings, when to do a blood glucose test, and where to apply a Sensor. Tap **NEXT** to proceed until you see the Scan New Sensor screen.

3 Use the App to scan your Sensor

How to Scan

Tap scan button  . You can either tap the blue box on the Home Screen or  at the top right. Hold top of iPhone near Sensor until you get a tone and/or vibration.


STEP 1



Scan your Sensor to start it.


Note: If you want to use both the Reader and the App to check your glucose, you must start the Sensor with the Reader first.

STEP 2

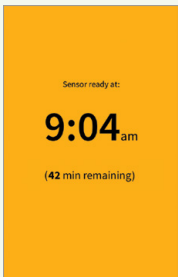

The Sensor can be used to check your glucose after the start-up period. While the Sensor is starting up, you can navigate away from the App.


STEP 3

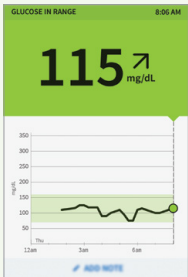

After the start-up period, scan your Sensor to get your glucose reading.

Glucose Graph

Graph of your current and stored glucose readings

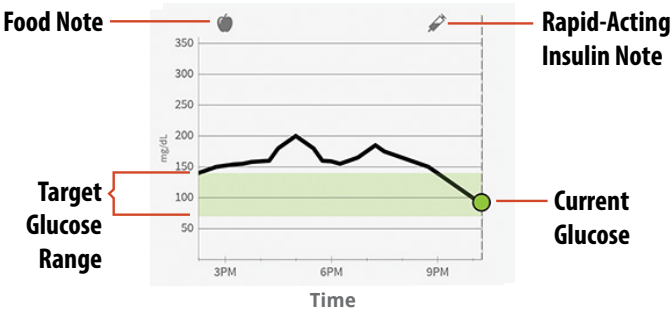

Message


Tap for more information

Check Blood Glucose

When you see this symbol, do a blood glucose test before making treatment decisions

Current Glucose

Glucose from your latest scan

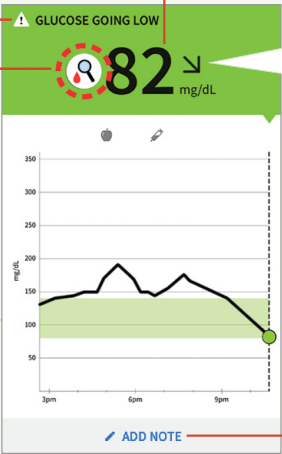


Add Note
Tap to add notes to glucose reading


Glucose Trend Arrow

Direction your glucose is going


Arrow	What it means
↑	Glucose rising quickly
↗	Glucose rising
→	Glucose changing slowly
↘	Glucose falling
↓	Glucose falling quickly

WARNING: When you see the  symbol, you must check your blood glucose with a blood glucose meter before making any treatment decisions. Sensor readings may not accurately reflect blood glucose levels.


Reports of your glucose history

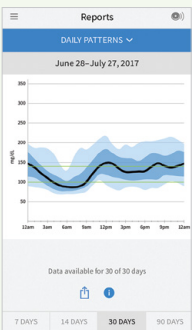


Daily Patterns

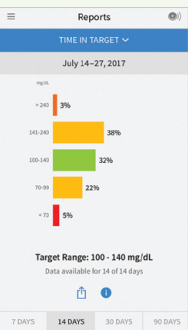


Time In Target

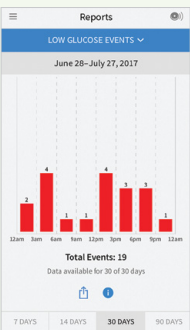


Low Glucose Events

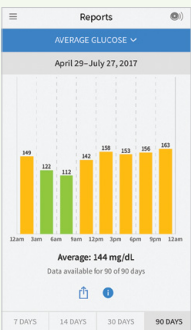

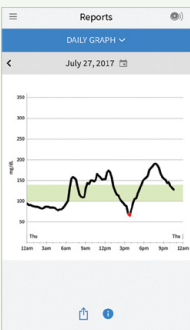


Average Glucose

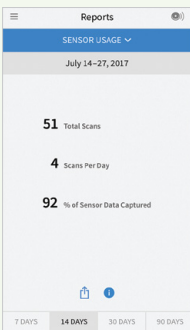

Daily Graph




Sensor Usage

- Tap  at the top left of the screen to access your reports.
- Tap  to share a screenshot of the report.
- Tap  to view a description of the report.

If you have followed the instructions described in the User's Manual and are still having difficulty setting up your System or if you are experiencing symptoms that are not consistent with your glucose monitoring, contact your health care professional.

FreeStyle, Libre, and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions.
©2018 Abbott ART38708-001 Rev. A 09/18

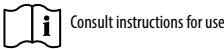

Manufacturer
Abbott Diabetes Care Inc.
1360 South Loop Road
Alameda, CA 94502 USA

