FreeStyle *LibreLink*

Smartphone Compatibility Guide

Click here for English

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FreeStyle LibreLink App

Smartphone Compatibility Guide

What smartphones and operating systems are compatible with the FreeStyle LibreLink app?

FreeStyle LibreLink will only work with phones that meet the following minimum requirements:

- Android Operating System 5.0 and higher, Near Field Communication (NFC) capability
- iPhone 7 and higher running iOS 11 and higher

Scan performance varies due to differences in phone hardware and operating systems. The phone/operating systems in Table 1 have been tested and are recommended for use with FreeStyle LibreLink and FreeStyle Libre sensors. The phone/operating systems in Table 2 have been tested and are not recommended for use with FreeStyle LibreLink and FreeStyle Libre sensors.

For phone/operating systems other than those listed, Abbott Diabetes Care has not assessed scan performance.

Table 1: Recommended phone/operating system combinations

DESCRIPTION	SMARTPHONE MODEL	OPERATING SYSTEM
Samsung Galaxy S5	SM-G900M, SM-G900i, SM-G900F	Android 6.0.1
	SM-G900W8	Android 5.0
	SM-G900T	Android 5.1.1
Samsung Galaxy S6	SM-G9208/SS	Android 5.1.1
	SM-G9209, SM-G920F	Android 6.0.1
	SM-G920F	Android 7.0
Samsung Galaxy S7 Edge	SM-G935U	Android 6.0.1
	SM-G935F	Android 7.0
Samsung Galaxy S7	SM-G930F	Android 6.0.1
Samsung Galaxy S8	SM-G950F	Android 7.0
Samsung Galaxy S8+	SM-G955F	Android 7.0
Huawei Nexus 6P	H1511	Android 6.0.1
Google Pixel	G-2PW4100	Android 7.1
LG Nexus 5X	LG-H791	Android 6.0
Samsung Galaxy S9	SM-6960U1	Android 8.0
Google Pixel 2	G011A	Android 8.1.0
Google Pixel 2 XL	G011C	Android 8.1.0
OnePlus 5T	A5010	Android 8.1.0
iPhone 7	A1660	iOS 11.1.1
iPhone 7 Plus	A1661	iOS 11.1
iPhone 8	A1863	iOS 11.1.2
iPhone 8 Plus	A1864	iOS 11.1.1
iPhone X	A1901	iOS 11.2.1

Table 2: Not recommended phone/operating system combinations

DESCRIPTION	SMARTPHONE MODEL	OPERATING SYSTEM
Samsung Galaxy S6	SM-G920A, SM-G920V, SM-G920T	Android 6.0.1

These tables will be updated as other phones and operating systems are evaluated.

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