- STEP 1:** Select site.  
**STEP 2:** Wash site using a plain soap, dry, and then clean with alcohol wipe. Allow site to air dry.  
**STEP 3:** Peel lid completely off Sensor Pack. Unscrew cap from Sensor Applicator.

**CAUTION:** Sensor codes must match on Sensor Pack and Sensor Applicator. Do NOT use if damaged or already opened.

- STEP 4:** Line up dark marks on Sensor Applicator and Sensor Pack. On a hard surface, press down firmly on Sensor Applicator until it comes to a stop.

- STEP 5:** Lift Sensor Applicator out of Sensor Pack.  
**STEP 6:** Sensor Applicator is ready to apply Sensor.

**CAUTION:** Sensor Applicator now contains a needle. Do not touch inside Sensor Applicator or put it back into Sensor Pack.

- STEP 7:** Place Sensor Applicator over site. Push down firmly to apply Sensor.

**CAUTION:** Do NOT push down on Sensor Applicator until placed over prepared site to prevent unintended results or injury.

- STEP 8:** Gently pull Sensor Applicator away.  
**STEP 9:** Make sure Sensor is secure. Discard the used Sensor Pack and Sensor Applicator. See *Disposal* section in the User's Manual.

To use your Sensor, refer to all the labelling provided with the Reader Kit. If using FreeStyle LibreLink app or FreeStyle LibreLink special edition app, refer to the User's Manual in the App.

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